## CORMAY BILE ACIDS CALIBRATOR

Nr kat. **3-125** (PL)  
**CORMAY BILE ACIDS CALIBRATOR** 1 x 2 ml

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Cat. No **3-125** (EN)  
**CORMAY BILE ACIDS CALIBRATOR** 1 x 2 ml

### INTENDED USE

Calibrator for determination of bile acid concentration.

The calibrator must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

### INTRODUCTION

Bile acids are the main product of degradation of endogenous cholesterol formed in the liver. Total bile acids are metabolized in the liver and are a valuable indicator of normal or abnormal liver function. Serum total bile acids are increased in patients with viral hepatitis, liver cirrhosis and liver cancer.

### COMPOSITION

CORMAY BILE ACIDS CALIBRATOR is single level liquid stable calibrator prepared in a buffer base.

The exact calibrator concentration is printed on the label of each vial.

### PREPARATION AND STORAGE

The calibrator is ready for use.

Unopened calibrator stored at 2-8°C is stable until expiry date given on the product label. After opening calibrator is stable for 30 days when stored at 2-8°C.

Store calibrator tightly capped when not in use.

### WARNINGS AND NOTES

- Protect from direct sunlight and contamination.
- Possible infectious agents in the materials have been inactivated. Because no method can offer complete assurance as to the absence of infectious agents, this material should be handled as though capable of transmitting infectious disease and disposed as biohazard waste or medical waste according to applicable local and national laws.

### TRACEABILITY

The calibrator's value is assigned referring to the Internal Standard.

### WASTE MANAGEMENT

Please refer to local legal requirements.

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