CORMAY BILE ACIDS CONTROLS

CONTROL SERUM FOR DETERMINATION OF BILE ACIDS CONCENTRATION

Kit name	Kit size	Cat. No
CORMAY BILE ACIDS CONTROLS	2 x 1 ml	5-149

CORMAY BILE ACIDS CONTROLS are designed for routine control of bile acids concentration at human serum. Kit contains lyophilized controls at two levels (low - L and high - H).

PACKAGE

	Size
BILE ACIDS CONTROL (L)	1 x 1 ml
BILE ACIDS CONTROL (H)	1 x 1 ml

PROCEDURE

- Open carefully the vial and add exactly 1 ml of deionized water
- Close bottle and leave to stand for 30 min. at room temperature.
- Swirl gently several times to ensure that the contents are completely dissolved.

STABILITY AND STORAGE

- Lyophilised serum remains stable when stored at -20°C until expiry date given on the product label.
- The constituents in reconstituted control serum are stable for 2 weeks at 2 - 8°C and 3 weeks at -20°C.

WARNINGS AND NOTES

- Product for in vitro diagnostic use only.
- Controls should be used only for the purpose intended by suitably qualified laboratory personnel, under appropriate laboratory conditions.
- Do not use controls after expiry date.
- Only one freezing of the reconstituted serum is recommended! Preparation of portions and freezing should be done immediately after reconstitution. Thaw at room temperature for about 30 minutes. Swirl gently several times during thawing.
- The controls are based on bovine serum that has been tested for possible infectious agents and found to be negative. However this material should be handled as though capable of transmitting infectious disease.

ASSIGNED VALUES

Assigned values, vary from lot to lot are given on the label of each vial.

WASTE MANAGEMENT

Please refer to local legal requirements.

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PZ CORMAY S.A.