

# CORMAY CK-MB CONTROL N CORMAY CK-MB CONTROL P



## CONTROL SERUM (LYOPHILISED) FOR DETERMINATION OF CK-MB ACTIVITY

CORMAY CK-MB CONTROL N	1 x 3 ml	Cat. No 5-183
CORMAY CK-MB CONTROL P	1 x 3 ml	Cat. No 5-184

CORMAY CK-MB CONTROL N and CORMAY CK-MB CONTROL P are lyophilisate prepared from human serum. They are designed for routine quality controls of CK-MB activity. The exact CK-MB values are printed on the label.

### PREPARATION

1. Remove vials from refrigerator and allow to warm to room temperature for 15 to 20 minutes.
2. Remove the seal and rubber stopper from vials. Volumetrically add exactly 3 ml of distilled or deionized water using. The water used for reconstitution should be at room temperature.
3. Carefully close bottle and let stand out for 30 min. ensure that all traces of dry material are dissolved by swirling gently.
4. Use immediately or refrigerate at 2-8°C for future use.

### STABILITY AND STORAGE

1. Lyophilised serum remains stable when stored at 2-8°C until expiry date given on the product label.
2. The constituents in reconstituted control serum are stable for 4 weeks at 2-8°C.
3. Protect from light and heat!

### NOTES

1. Product for in vitro diagnostic use only.
2. Control serum must be used only for the purpose intended by suitably qualified laboratory personnel, under appropriate laboratory conditions.
3. This control serum has been tested for the HIV antibody, HBsAg and HCV and found to be non-reactive. However this material should be handled as though capable of transmitting infectious disease.

### WASTE MANAGEMENT

Please refer to local legal requirements.

### LITERATURE:

1. Gerhardt W., et al. Clin Chem Acta 78:29 (1977).
2. Giegel J.L., et al. Clin Chem 28:1364 (1982).
3. Mercer D.W., Clin Chem 20:36 (1974).
4. Neumetier D., et al. Clin Chem Acta 73:445 (1976).
5. Roberts R., et al. The Lancet 319 (1977).
6. Vaidya H.C., et al. Clin Chem 32:657 (1986).
7. Wicks R.W., et al. Clin. Chem 28:54 (1982).
8. Willerson J.T., et al. Proc Natl Acad Sci USA 74:1711 (1977).

**Date of issue:** 02. 2012.

### MANUFACTURER

**PZ CORMAY S.A.**  
ul. Wiosenna 22,  
05-092 Łomianki, POLAND  
tel.: +48 (0) 22 751 79 10  
fax: +48 (0) 22 751 79 14  
<http://www.cormay.pl>