ACCENT-200 BIL DIRECT

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(EN)

2-REAGENT

sodium metavanadate 4.1 mmol/l
phosphate buffer
pH adjuster
stabilizer

REAGENT STABILITY

The reagents when stored at 2-8°C are stable up to expiry date printed on the package. The reagents are stable for 12 weeks on board the analyser at 2-10°C (ACCENT-200 (II GEN), ACCENT MC240, ACCENT S120).

WARNINGS AND NOTES

- Protect from direct sunlight and avoid contamination!
- Do not freeze reagents.
- Do not interchange caps.
- Reagent bottles should be shaken before use by gently inverting several times.
- The appearance of turbidity or control sera values outside the manufacturer's acceptable range may indicate of the reagents instability.
- Lack of significant changes in the color of the reaction mixture at the samples with low bilirubin concentration does not indicate the assay malfunction.
- Do not use the reagent beyond expiry date printed on the package.
- Do not mix reagents from different kits or lots.
- Use personal protective equipment to prevent contact with samples, reagents and controls.
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.
- 1-REAGENT meets the criteria for classification in accordance with Regulation (EC) No 1272/2008.

Warning

 H319 Causes serious eye irritation.
EUH208 Contains hydroxylammonium chloride.
May produce an allergic reaction.

P280 Wear protective gloves, protective clothing, eye protection or face protection.

P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

SPECIMEN^{6,7,8}

Serum free from hemolysis.
Serum should be separated from red blood cells as soon as possible after blood collection.

Lipemic specimens may show falsely decreased bilirubin concentration thus fasting specimen is recommended.

Because bilirubin is photooxidized when exposed to light, specimen should be protected from direct exposure to either artificial light or sunlight. Therefore it is essential to store specimens in the dark! Serum can be stored at 2-8°C or at room temperature (20-25°C) up to 2 days.

Nevertheless it is recommended to perform the assay with freshly collected samples!

INTENDED USE

BIL DIRECT reagent is intended to determine quantitatively direct bilirubin level in serum. It is intended to diagnosis, monitoring and as an aid to diagnosis of clinical conditions connected with abnormal bilirubin level. BIL DIRECT reagent is intended to use on automatic analysers ACCENT-200 (II GEN) / BS-200, ACCENT-220S / BS-180, ACCENT S120 / BS-230, ACCENT MC240 / BS-240Pro, ACCENT M320 / BS-360E, BS-120, ACCENT 400 and ACCENT Neo200. It is only for *in vitro* diagnostics, for healthcare professional users.

SUMMARY^{1,2,3}

Determinations of direct bilirubin level in serum are mainly used in diagnosis of liver functions. Measurement of direct bilirubin concentration is used as an aid in diagnosis of clinical conditions associated with hyperbilirubinemia. Increased direct bilirubin levels can be observed in mechanical jaundice, Dubin-Johnson syndrome, bile ducts diseases, gallbladder diseases. Direct bilirubin is also the marker of perforation in acute sigmoid diverticulitis.

METHOD PRINCIPLE^{4,5}

Method is based on chemical oxidation, utilizing vanadate as an oxidizing agent.

In the presence of detergent and and vanadate in an acidic solution, conjugated (direct) bilirubin is oxidized to produce biliverdin.

This oxidation reaction causes change of the yellow colour, which is specific to bilirubin to the green colour typical for biliverdin. Therefore, the direct bilirubin concentration in the sample can be obtained by measuring the absorbance before and after the vanadate oxidation.

REAGENTS

Package

1-REAGENT 2 x 26 ml
2-REAGENT 1 x 13.5 ml

Number of tests:

ACCENT-200 (II GEN) 160
ACCENT-220S 160
ACCENT S120 220
ACCENT MC240 220
ACCENT M320 220
BS-120 160

CONCENTRATION OF THE ACTIVE INGREDIENTS IN THE REAGENT

1-REAGENT

citrate buffer
hydroxylammonium chloride
stabilizers
detergent 0.34 %
pH adjuster
preservative

ACCENT-200 BIL DIRECT (IV GENERACJA / IV GENERATION / IV ПОКОЛЕНИЕ)
51_03_03_015_06

ACCENT-200 BIL DIRECT (IV GENERACJA / IV GENERATION / IV ПОКОЛЕНИЕ)
51_03_03_015_06

str. / page / стр. 3/14

str. / page / стр. 4/14

Follow tube manufacturers' instructions carefully when using collection tubes.

Human-origin material should be handled as potentially infectious. Standard precautions in normal laboratory work are required.

ASSAY PROCEDURE

1-REAGENT and 2-REAGENT are ready to use.

For reagent blank deionized water is recommended.

For analyzers: ACCENT-200 (II GEN), ACCENT-220S and BS-120, it is recommended to determine the reagent blank during each calibration. Deionized water should be used as reagent blank. When performing calibration, the task type **Calib+Rgt.Blk** should be selected.

Actions required:

When performing assays in analyzers: ACCENT-200 (II GEN), ACCENT-220S and BS-120 there is a probability of **cross-contamination** affecting the tests results: CREATININE - BIL DIRECT. To avoid this effect follow the recommendations contained in the instruction 51_03_24_001_ACCENT 200_CARRYOVER.

QUALITY CONTROL

For internal quality control it is recommended to use with each batch of samples, the CORMAY SERUM HN (Cat. No 5-172) and CORMAY SERUM HP (Cat. No 5-173). For the calibration of automatic analysers the CORMAY MULTICALIBRATOR LEVEL 1 (Cat. No 5-174; 5-176) and LEVEL 2 (Cat. No 5-175; 5-177) are recommended. Deionised water should be used as a calibrator 0.

The calibration curve should be prepared every 12 weeks (ACCENT-200 (II GEN), every 11 weeks (ACCENT MC240) or every 8 weeks (ACCENT S120).

Calibration is recommended in the following cases:

- after each change of lot,
- after instrument service,
- if controls lie outside the expected range,
- each time a new reagent kit is used.

If the quality control results do not fall within the expected values or within the range determined in the laboratory, despite a successful calibration procedure, do not report results. In this case, please take the following actions:

- verify reagents are not out of expiration date.
- verify that the required maintenance has been carried out.
- verify that the procedure has been performed in accordance with the instructions for use.
- contact the Service Department or distributor for assistance.

REFERENCE VALUES ³

serum (adults)	0.1 – 0.3 mg/dl
	1.7 – 5.1 µmol/l

It is recommended for each laboratory to establish its own reference ranges for local population and method principle.

The assay results should be used in conjunction with other data, such as symptoms, results of other tests and clinical history to make clinical decisions. It is not recommended to make clinical diagnosis based on a single result.

PERFORMANCE CHARACTERISTICS

The following results have been obtained using automatic analysers ACCENT-200 (II GEN) and/or ACCENT MC240 and/or BS-400. Results may vary if a different instrument is used.

- LoB (Limit of Blank):**
0.00 mg/dl (0.00 µmol/l) – BS-400
- LoD (Limit of Detection):**
0.01 mg/dl (0.171 µmol/l) – BS-400
- LoQ (Limit of Quantitation):**
0.09 mg/dl (1.539 µmol/l) – ACCENT-200 (II GEN)
0.04 mg/dl (0.684 µmol/l) – ACCENT MC240
- Linearity:**
up to 40 mg/dl (684 µmol/l) – ACCENT-200 (II GEN);
ACCENT MC240
- Measurement range:**
0.09 mg/dl (1.539 µmol/l) – 40 mg/dl (684 µmol/l) – ACCENT-200 (II GEN)
0.04 mg/dl (0.684 µmol/l) – 40 mg/dl (684 µmol/l) – ACCENT MC240

Specificity / Interferences

Ascorbic acid up to 62 mg/l and triglycerides up to 550 mg/dl (in samples containing low concentration of direct bilirubin) and up to 900 mg/dl (in samples containing high concentration of direct bilirubin) do not interfere with the test. Haemoglobin interferes even in small amount with the determination.

Precision

Repeatability (run to run)		Mean [mg/dl]	SD [mg/dl]	CV [%]
ACCENT MC240 (n = 20)	poziom 1	0.35	0.01	1.9
	poziom 2	1.04	0.01	0.8
ACCENT-200 (II GEN) (n = 80)	poziom 1	0.31	0.01	4.1
	poziom 2	1.00	0.02	1.6
Reproducibility (day to day)		Mean [mg/dl]	SD [mg/dl]	CV [%]
ACCENT MC-240 (n = 80)	poziom 1	0.37	0.01	3.5
	poziom 2	1.10	0.06	5.5
ACCENT-200 (II GEN) (n = 80)	poziom 1	0.33	0.02	4.7
	poziom 2	1.00	0.04	3.8

Method comparison

A comparison between direct bilirubin values determined at **ACCENT-200 (II GEN)** (y) and at **BECKMAN COULTER AU680** (x) using 60 serum samples gave following results:

$$y = 1.0789x - 0.0496 \text{ mg/dl}$$

$$R = 0.999 \quad (R - \text{correlation coefficient})$$

A comparison between direct bilirubin values determined at **ACCENT MC240** (y) and at **BECKMAN COULTER AU680** (x) using 119 serum samples gave following results:

$$y = 1.0987x - 0.0288 \text{ mg/dl}$$

$$R = 0.999 \quad (R - \text{correlation coefficient})$$

WASTE MANAGEMENT⁹

After use, the reagents should be handled as potentially infectious and disposed of in accordance with local legal requirements.

- Residual reagents:
1-REAGENT – 18 01 06*
2-REAGENT – 18 01 07
- Empty packages:
1-REAGENT – 15 01 10*
2-REAGENT – 15 01 02
- Wastewater from the analyzer: 18 01 03*

INCIDENTS¹⁰

Any serious incident that has occurred in relations to the device shall be reported to the manufacturer (website address: incidents@cornay.pl) and the competent authority of the Member State in which the user and/or the patient is established.

Serious incident means any incident that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- a serious public health threat.

LITERATURE

- Rifai N., Horvath A.R., Wittwer C., ed. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics 6th ed., St. Louis, Missouri, Elsevier, 767-771 (2018).
- Käser S.A., Fankhauser G., Glauser P.M. et al. Diagnostic Value of Inflammation Markers in Predicting Perforation in Acute Sigmoid Diverticulitis. World J Surg 34, 2717–2722 (2010).
- Pagana K, Pagana T, Pagana T. Mosby's Diagnostic & Laboratory Test Reference 14th ed., St. Louis, Missouri, Elsevier, 137-139 (2019).
- Tokuda K., Tanimoto K. A new method of measuring bilirubin in serum by vanadic acid. Jpn J Clin Chem (in Japanese with English abstract) 22: 116–22 (1993).
- Dhungana N., Morris C., Krasowski M.D. Operational impact of using a vanadate oxidase method for direct bilirubin measurements at an academic medical center clinical laboratory. Practical laboratory medicine 8: 77-85 (2017).
- Tietz N.W., ed. Clinical Guide to Laboratory Test. 3rd ed. Philadelphia: WB Saunders 518-525 (1995).
- Kaplan L.A., Pesce A.J., Clinical Chemistry: Theory, analysis, and correlation, 2nd ed., The C.V. Mosby Company, St. Louis, USA, 1013 (1989).
- World Health Organization (2002). Use of Anticoagulants in Diagnostic Laboratory Investigations (Stability of Blood, Plasma and serum samples); WHO/DIL/LAB/99.1 Rev.2: 1-64.
- European Commission notice on technical guidance on the classification of waste (2018/C 124/01) of 9 April 2018.
- Regulation (EU) 2017/746 of the European Parliament and of the Council of 5 April 2017 on in vitro diagnostic medical devices.

LIST OF CHANGES

Previous version: 05	Current version: 06
<i>Sections updated: WARNING AND NOTES; WASTE MANAGEMENT</i>	

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1. Rifai N., Horvath A.R., Wittwer C., ed. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics 6th ed., St. Louis, Missouri, Elsevier, 767-771 (2018).
2. Käser S.A., Fankhauser G., Glauser P.M. et al. Diagnostic Value of Inflammation Markers in Predicting Perforation in Acute Sigmoid Diverticulitis. World J Surg 34, 2717-2722 (2010).
3. Pagana K, Pagana T, Pagana T. Mosby's Diagnostic & Laboratory Test Reference 14th ed., St. Louis, Missouri, Elsevier, 137-139 (2019).
4. Tokuda K., Tanimoto K. A new method of measuring bilirubin in serum by vanadic acid. Jpn J Clin Chem (in Japanese with English abstract) 22: 116-22 (1993).
5. Dhungana N., Morris C., Krasowski M.D. Operational impact of using a vanadate oxidase method for direct bilirubin measurements at an academic medical center clinical laboratory. Practical laboratory medicine 8: 77-85 (2017).
6. Tietz N.W., ed. Clinical Guide to Laboratory Test. 3rd ed. Philadelphia: WB Saunders 518-525 (1995).
7. Kaplan L.A., Pesce A.J., Clinical Chemistry: Theory, analysis, and correlation, 2nd ed., The C.V. Mosby Company, St. Louis, USA, 1013 (1989).
8. World Health Organization (2002). Use of Anticoagulants in Diagnostic Laboratory Investigations (Stability of Blood, Plasma and serum samples); WHO/DIL/LAB/99.1 Rev.2: 1-64.
9. European Commission notice on technical guidance on the classification of waste (2018/C 124/01) of 9 April 2018.
10. Regulation (EU) 2017/746 of the European Parliament and of the Council of 5 April 2017 on in vitro diagnostic medical devices.

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Calibration Rule

Rule	Multi-point Linear
Sensitivity	1
Replicates	2
Interval (day)	84
Difference Limit	0
SD	0
Blank Response	0 50000
Error Limit	0
Coefficient	0

• ACCENT-220S

Parameters

Test	BIL D	R1	280
No	7	R2	70
Full Name	Bilirubin Direct	Sample Volume	10
Standard No	7	R1 Blank	
Reac. Type	Endpoint	Mixed Rtg. Blank	
Pri. Wave.	450 nm	Linearity Range	0.08 40
Sec. Wave.	546 nm	Linearity Limit	
Direction	Decrease	Substrate Limit	
Reac. Time	-1 20	Factor	
Incuba. Time	15	<input type="checkbox"/> Prozone check	
Unit	mg/dl	q1 <input type="checkbox"/> q2 <input type="checkbox"/> q3 <input type="checkbox"/> q4 <input type="checkbox"/>	
Precision	0.01	PC <input type="checkbox"/> Abs <input type="checkbox"/>	

Calibration Rule

Rule	Multi-point Linear
Sensitivity	1
Replicates	2
Interval (day)	56
Difference Limit	0
SD	0
Blank Response	0 50000
Error Limit	0
Coefficient	0

• BS-120

Parameters

Test	BIL D	R1	280
No	7	R2	70
Full Name	Bil Direct	Sample Volume	10
Standard No	7	R1 Blank	
Reac. Type	Endpoint	Mixed Rtg. Blank	
Pri. Wave.	450 nm	Linearity Range	0.09 40
Sec. Wave.	546 nm	Linearity Limit	
Direction	Descending	Substrate Limit	
Reac. Time	-1 16	Factor	
Incuba. Time	16	<input type="checkbox"/> Prozone check	
Unit	mg/dl	q1 <input type="checkbox"/> q2 <input type="checkbox"/> q3 <input type="checkbox"/> q4 <input type="checkbox"/>	
Precision	0.01	PC <input type="checkbox"/> Abs <input type="checkbox"/>	

Calibration Rule

Rule	Multi-point Linear
Sensitivity	1
Replicates	2
Interval (day)	56
Difference Limit	0
SD	0
Blank Response	0 50000
Error Limit	0
Coefficient	0

• ACCENT S120

Chem	<input type="text" value="BIL DIRECT"/>	No.	<input type="text" value="007"/>	Sample Type	<input type="text" value="SERUM"/>
Chemistry	<input type="text" value="BILIRUBIN DIRECT"/>	Print name	<input type="text" value="BIL D"/>		
Reaction Type	<input type="text" value="Endpoint"/>	Reaction Direction	<input type="text" value="negative"/>		
Pri Wave	<input type="text" value="450 nm"/>	Sec Wave	<input type="text" value="546 nm"/>		
Unit	<input type="text" value="mg/dl"/>	Decimal	<input type="text" value="0.01"/>		
Blank Time	<input type="text" value="-3"/>	<input type="text" value="-1"/>			
Standard	Sample Vol <input type="text" value="7"/> μ L	Aspirated <input type="text" value=""/>	Diluent <input type="text" value=""/>	Reagent Vol <input type="text" value="200"/> μ L	
Decreased	<input type="text" value="7"/> μ L	<input type="text" value="20"/> μ L	<input type="text" value="180"/> μ L	R1 <input type="text" value="50"/> μ L	
Increased	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	R2 <input type="text" value=""/>	
	<input type="checkbox"/> Sample Blank	<input checked="" type="checkbox"/> Auto Rerun			

Linearity range (Standard)	<input type="text" value="0.09"/>	<input type="text" value="40"/>	Linearity Limit	<input type="text" value=""/>
Linearity Range (Decreased)	<input type="text" value=""/>	<input type="text" value=""/>	Substrate Depletion	<input type="text" value=""/>
Linearity Range (Increased)	<input type="text" value=""/>	<input type="text" value=""/>	Mixed Blank Abs	<input type="text" value="-40000"/> <input type="text" value="40000"/>
R1 Blank Abs	<input type="text" value="-40000"/>	<input type="text" value="40000"/>	On-board Stability	<input type="text" value="84"/> Day(s)
Blank Response	<input type="text" value="-40000"/>	<input type="text" value="40000"/>	Reagent Alarm Limit	<input type="text" value=""/>
Twin Chemistry	<input type="text" value=""/>	<input type="text" value=""/>	<input type="checkbox"/> Enzyme Linear Extension	
<input type="checkbox"/> Prozone Check				
O1 <input type="text" value=""/>	O2 <input type="text" value=""/>	V1 <input type="text" value=""/>	O3 <input type="text" value=""/>	O4 <input type="text" value=""/>
Q5 <input type="text" value=""/>	Q6 <input type="text" value=""/>	V3 <input type="text" value=""/>	PC1 <input type="text" value=""/>	PC2 <input type="text" value=""/>
<input type="checkbox"/> Sample Pretreatment	<input type="checkbox"/> Control Pretreatment	<input type="checkbox"/> Calibrator Pretreatment		
	Pretreat Sample Vol <input type="text" value=""/>	μ L	Pretreat Sample Vol <input type="text" value=""/>	μ L

CALIBRATION SETTINGS		AUTO CALIBRATION	
Math model	<input type="text" value="Multi-point linear"/>	<input type="checkbox"/> Bottle Changed	
Factor	<input type="text" value=""/>	<input type="checkbox"/> Lot Changed	
Replicates	<input type="text" value="2"/>	<input type="checkbox"/> Cal Time	

ACCEPTANCE LIMITS	
Cal Time	<input type="text" value="1344"/> Hour
Slope Diff	<input type="text" value=""/>
Sensitivity	<input type="text" value=""/>
Deter Coeff	<input type="text" value=""/>

• ACCENT MC240

Chem	<input type="text" value="BIL DIRECT"/>	No.	<input type="text" value="007"/>	Sample Type	<input type="text" value="SERUM"/>
Chemistry	<input type="text" value="BILIRUBIN DIRECT"/>	Print name	<input type="text" value="BIL DIRECT"/>		
Reaction Type	<input type="text" value="Endpoint"/>	Reaction Direction	<input type="text" value="negative"/>		
Pri Wave	<input type="text" value="450 nm"/>	Sec Wave	<input type="text" value="546 nm"/>		
Unit	<input type="text" value="mg/dl"/>	Decimal	<input type="text" value="0.01"/>		
Blank Time	<input type="text" value="-3"/>	<input type="text" value="-1"/>			
Standard	Sample Vol <input type="text" value="7"/> μ L	Aspirated <input type="text" value=""/>	Diluent <input type="text" value=""/>	Reagent Vol <input type="text" value="200"/> μ L	
Decreased	<input type="text" value="7"/> μ L	<input type="text" value="20"/> μ L	<input type="text" value="180"/> μ L	R1 <input type="text" value="50"/> μ L	
Increased	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	R2 <input type="text" value=""/>	
	<input type="checkbox"/> Sample Blank	<input checked="" type="checkbox"/> Auto Rerun			

Linearity range (Standard)	<input type="text" value="0.04"/>	<input type="text" value="40"/>	Linearity Limit	<input type="text" value=""/>
Linearity Range (Decreased)	<input type="text" value=""/>	<input type="text" value=""/>	Substrate Depletion	<input type="text" value=""/>
Linearity Range (Increased)	<input type="text" value=""/>	<input type="text" value=""/>	Mixed Blank Abs	<input type="text" value="-35000"/> <input type="text" value="35000"/>
R1 Blank Abs	<input type="text" value="-35000"/>	<input type="text" value="35000"/>	On-board Stability	<input type="text" value="84"/> Day(s)
Blank Response	<input type="text" value="-35000"/>	<input type="text" value="35000"/>	Reagent Alarm Limit	<input type="text" value=""/>
Twin Chemistry	<input type="text" value=""/>	<input type="text" value=""/>	<input type="checkbox"/> Enzyme Linear Extension	
<input type="checkbox"/> Prozone Check				
Q1 <input type="text" value=""/>	Q2 <input type="text" value=""/>	V1 <input type="text" value=""/>	Q3 <input type="text" value=""/>	Q4 <input type="text" value=""/>
O5 <input type="text" value=""/>	O6 <input type="text" value=""/>	V3 <input type="text" value=""/>	PC1 <input type="text" value=""/>	PC2 <input type="text" value=""/>
<input type="checkbox"/> Sample Pretreatment	<input type="checkbox"/> Control Pretreatment	<input type="checkbox"/> Calibrator Pretreatment		
	Pretreat Sample Vol <input type="text" value=""/>	μ L	Pretreat Sample Vol <input type="text" value=""/>	μ L

CALIBRATION SETTINGS		AUTO CALIBRATION	
Math model	<input type="text" value="Multi-point linear"/>	<input type="checkbox"/> Bottle Changed	
Factor	<input type="text" value=""/>	<input type="checkbox"/> Lot Changed	
Replicates	<input type="text" value="2"/>	<input type="checkbox"/> Cal Time	

ACCEPTANCE LIMITS	
Cal Time	<input type="text" value="1848"/> Hour
Slope Diff	<input type="text" value=""/>
Sensitivity	<input type="text" value=""/>
Deter Coeff	<input type="text" value=""/>

• ACCENT M320

Chem	BIL DIRECT		No.	007		Sample Type	SERUM	
Chemistry	BILIRUBIN DIRECT		Print name	BIL D		Reaction Direction	negative	
Reaction Type	Endpoint		Sec Wave	546 nm		Incubation Time	20	
Pri Wave	450 nm		Decimal	0.01		Reaction Time	28 30	
Unit	mg/dl		Blank Time	-3 -1		Sample Vol	Aspirated Diluent	
Standard	7	μL	Decreased	7	μL	Increased	20	μL
Decreased	7	μL	Increased	20	μL	R1	200	μL
Increased		μL	R2	50	μL	Sample Blank <input type="checkbox"/> Auto Rerun <input checked="" type="checkbox"/>		

Linearity range (Standard)	0.08	40	Linearity Limit	
Linearity Range (Decreased)			Substrate Depletion	
Linearity Range (Increased)			Mixed Blank Abs	-35000 35000
R1 Blank Abs	-35000	35000	On-board Stability	84 Day(s)
Blank Response	-35000	35000	Reagent Alarm Limit	
Twin Chemistry			Enzyme Linear Extension	<input type="checkbox"/>
<input type="checkbox"/> Prozone Check				
Q1	Q2	V1	Q3	Q4
Q5	Q6	V3	PC1	PC2
<input type="checkbox"/> Sample Pretreatment		<input type="checkbox"/> Control Pretreatment		<input type="checkbox"/> Calibrator Pretreatment
Pretreat Sample Vol		Pretreat Sample Vol		Pretreat Sample Vol

CALIBRATION SETTINGS

Math model	Multi-point linear
Factor	Replicates 2

AUTO CALIBRATION

<input type="checkbox"/>	Bottle Changed
<input type="checkbox"/>	Lot Changed
<input type="checkbox"/>	Cal Time

ACCEPTANCE LIMITS

Cal Time	1848 Hour
Slope Diff	SD
Sensitivity	Repeatability 35000
Deter Coeff	

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