## ACCENT-200 ETHANOL

Nr kat. **7-278** (PL)

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### LITERATURA

- Burtis C.A., Ashwood E.R., ed. Tietz Textbook of Clinical Chemistry, Toronto: WB Saunders Co., 1171-1175 (1994).
- William H., Porter Ph.D. Clinical Toxicology. In: Burtis CA, Ashwood ER, editors. Tietz Textbook of Clinical Chemistry. 3rd ed. Philadelphia: W.B Saunders Company; 1999. p. 922-923.
- Bonnichsen, R.K.,Theorell, H., An Enzymatic Method for the Micro Determination of Ethanol, Scand. J. Clin. Lab. Invest. 3, 58 (1951).
- Lundquist, F., The Determination of Ethyl Alcohol in Blood and Tissues., Methods of Biochemical Analysis, Vol III, Editor, D. Glick, Interscience Publishers, New York, 1957 p. 217-251.
- CLSI Guidelines and Standards, Clinical Laboratory Tests, AACC Press, Washington, 1990.
- Preeti Dalwari: [medicine.medscape.com/article/2090019-overview#a3](http://medicine.medscape.com/article/2090019-overview#a3).
- Burtis C.A., Ashwood E.R., ed. Tietz Textbook of Clinical Chemistry, 3rd ed. Philadelphia, PA: WB Saunders, 923, (1999).
- Krajowa Izba Diagnostów Laboratoryjnych, Oznaczanie etanolu w krwi i moczu pacjentów w laboratorium medycznym, Warszawa 2016. p.18-19.

Data wydania: 05. 2022.

## ACCENT-200 ETHANOL

Cat. No 7-278

(EN)

### INTENDED USE

Diagnostic kit for determination of ethyl alcohol concentration intended to use in automatic analyzers: ACCENT-200, ACCENT-200 II GEN, ACCENT-220S, ACCENT S120, ACCENT MC240, ACCENT M320, ACCENT 400 and ACCENT Neo200.

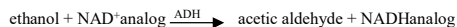
The reagents must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

### INTRODUCTION

Ethanol after consumption is absorbed in alimentary tract and transported with blood to whole organism. A little amount of ethanol is excreted with urine or exhaled with air, but majority is metabolized by liver to acetic aldehyde, then to acetate, finally to carbon dioxide and water. Ethanol consumption is often cause of various types of accidents, and ethanol poisoning may be cause of decease.

### METHOD PRINCIPLE

Enzymatic method with alcohol dehydrogenase.

The rate of absorbance changes measured at  $\lambda=340$  nm or  $\lambda=380$  nm is proportional to the quantity of ethyl alcohol in the sample.

### REAGENTS

#### Package

1-Reagent 2 x 35 ml

The reagent, stored tightly capped at 2-8°C is stable up to expiry date printed on the package. The reagent, stored on board of the analyser at 2-10°C is stable for: 5 weeks, (ACCENT-200) or 12 weeks (ACCENT M320).

### Concentrations in the test

buffer (pH 7,3)

NAD analog 1.35 mmol/l  
 alcohol dehydrogenase >203 kU/l  
 stabilizers, preservatives

### Warnings and notes

- Protect from direct sunlight and avoid contamination !
- Avoid contact with skin and mucous membranes.
- Concentration of ethyl alcohol in serum or plasma is approximately 1.2 times higher than in whole blood.<sup>6</sup>
- Turbidity would indicate contamination or deterioration of reagent.

### SPECIMEN

Serum or plasma (heparin, EDTA or citrate) free from hemolysis; urine.

Samples may be stored tightly closed in temp. 2-8°C up to 3 days. Urine samples should be stored with as little dead air space as possible.

Nevertheless it is recommended to perform the assay with freshly collected samples!

- Do not use ethanol to swab venipuncture site and to clean or sterilize glassware or other equipment used to collect the specimen or perform the assay.
- Samples should be at room temperature for testing.
- Before opening mix samples by inverting the container.

### PROCEDURE

1-Reagent is ready to use.

Deionised water or 0.9% NaCl is recommended as a reagent blank.

### REFERENCE VALUES <sup>7</sup>

Ethanol consumption cause following symptoms:

blood ethanol concentration (mg / dl)	clinical symptoms*
30-120	euphoria; sociability; talkativeness; diminution of attention, judgment and control; some sensory-motor impairment
90-250	emotional instability; loss of critical judgment; impairment of perception, memory and comprehension; increased reaction time; sensory-motor incoordination; drowsiness;
180-300	disorientation; mental confusion; dizziness; staggering gait; slurred speech; disturbances of vision; increased pain threshold; apathy; lethargy;
250-400	general inertia; markedly decreased response to stimuli; vomiting; incontinence of urine and feces; sleep or stupor;
350-500	complete unconsciousness; coma; depressed or abolished reflexes; impairment of circulation and respiration; possible death.

\*Effects are dependent on individual use, chronicity, tolerance and other factors.<sup>8</sup>

To calculate the concentration of ethyl alcohol in blood the result received for serum or plasma should be divided by 1.2 factor. Otherwise, doctor should be informed that the result is overstated and requires a proper interpretation.<sup>8</sup>

Please refer to the local regulations concerning legal ethanol limits for drivers.

Conversion factor of the blood ethanol concentration: 100 mg/dl = 1 ‰.

### QUALITY CONTROL

For internal quality control it is recommended to use, with each batch of samples, the CORMAY AMMONIA/ETHANOL CONTROLS (Cat. No 5-163).

For the calibration of automatic analysers the CORMAY ETHANOL CALIBRATORS (Cat. No 5-105) is recommended. Two levels of calibrator and deionised water

or 0.9% NaCl (as a calibrator 0) should be used for calibration.

The calibration curve should be prepared every 5 weeks (ACCENT-200) or every 12 weeks (ACCENT M320), with change of reagent lot number or as required e.g. quality control findings outside the specified range.

### PERFORMANCE CHARACTERISTICS

The following results have been obtained using the automatic analysers: ACCENT-200 or ACCENT-220S or ACCENT MC240 and Beckman CX. Results may vary if a different instrument or a manual procedure is used.

#### LoB (Limit of Blank):

1.9 mg/dl (0.41 mmol/l) – ACCENT-200  
 0.7 mg/dl (0.15 mmol/l) – ACCENT MC240

#### LoD (Limit of Detection):

3.7 mg/dl (0.80 mmol/l) – ACCENT-200  
 2.6 mg/dl (0.56 mmol/l) – ACCENT MC240

#### LoQ (Limit of Quantitation):

13.5 mg/dl (2.93 mmol/l) – ACCENT-200  
 7.0 mg/dl (1.52 mmol/l) – ACCENT MC240

#### Linearity:

up to 800 mg/dl (173.6 mmol/l) – ACCENT-200  
 up to 900 mg/dl (195.3 mmol/l) – ACCENT MC240

#### Specificity / Interferences

Haemoglobin up to 0.6 g/dl, bilirubin up to 40 mg/dl, intralipid up to 1000 mg/dl, LDH up to 10 000 U/l (in the presence of 200 mg/dl lactate) do not interfere with the test.

Acetone up to 2000 mg/dl, butanol up to 200 mg/dl, ethylene glycol up to 2000 mg/dl, methanol up to 2000 mg/dl, izopropanol up to 2000 mg/dl do not interfere with the test, which confirms that assay is specific to ethanol.

#### Precision

Repeatability (run to run)		Mean [mg/dl]	SD [mg/dl]	CV [%]
ACCENT-220S n=20	level 1	35.8	0.86	2.39
	level 2	89.5	1.48	1.66
	level 3	237.4	2.23	0.94
ACCENT MC240 n=20	level 1	35.6	1.10	3.09
	level 2	88.5	2.08	2.35
	level 3	222.4	4.95	2.23
Reproducibility (day to day)		Mean [mg/dl]	SD [mg/dl]	CV [%]
ACCENT-220S n=20	level 1	40.98	1.48	3.60
	level 2	100.64	2.68	2.67
	level 3	257.34	3.92	1.52
ACCENT MC240 n=80	level 1	39.5	2.25	5.7
	level 2	97.7	2.69	2.8
	level 3	246.1	6.47	2.6

#### Method comparison

A comparison between CORMAY reagent (y) and commercially available assay (x) using 40 serum samples, gave following results:

$$y = 1.064x - 2.33 \text{ mg/dl;}$$

$$R = 0.9994 \quad (R - \text{correlation coefficient})$$

### WASTE MANAGEMENT

Please refer to local legal requirements.

### LITERATURE

- Burtis C.A., Ashwood E.R., ed. Tietz Textbook of Clinical Chemistry, Toronto: WB Saunders Co., 1171-1175 (1994).
- William H., Porter Ph.D. Clinical Toxicology. In: Burtis CA, Ashwood ER, editors. Tietz Textbook of Clinical Chemistry. 3rd ed. Philadelphia: W.B Saunders Company; 1999. p. 922-923.
- Bonnichsen, R.K., Theorell, H., An Enzymatic Method for the Micro Determination of Ethanol, Scand. J. Clin. Lab. Invest. 3, 58 (1951).
- Lundquist, F., The Determination of Ethyl Alcohol in Blood and Tissues., Methods of Biochemical Analysis, Vol III, Editor, D. Glick, Interscience Publishers, New York, 1957 p. 217-251.
- CLSI Guidelines and Standards, Clinical Laboratory Repts, AAC Press, Washington, 1990.
- Preeti Dalawari: emedicine.medscape.com/article/2090019-overview#a3.
- Burtis C.A., Ashwood E.R., ed. Tietz Textbook of Clinical Chemistry, 3rd ed. Philadelphia, PA: WB Saunders, 923, (1999).
- Polish National Chamber of Laboratory Diagnosticians, Oznaczenie etanolu w krwi i moczu pacjentów w laboratorium medycznym, Warszawa 2016. p.18-19.

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- Burtis C.A., Ashwood E.R., ed. Tietz Textbook of Clinical Chemistry, Toronto: WB Saunders Co., 1171-1175 (1994).
- William H., Porter Ph.D. Clinical Toxicology. In: Burtis CA, Ashwood ER, editors. Tietz Textbook of Clinical Chemistry. 3rd ed. Philadelphia: W.B Saunders Company; 1999. p. 922-923.
- Bonnichsen, R.K., Theorell, H., An Enzymatic Method for the Micro Determination of Ethanol, Scand. J. Clin. Lab. Invest. 3, 58 (1951).
- Lundquist, F., The Determination of Ethyl Alcohol in Blood and Tissues., Methods of Biochemical Analysis, Vol III, Editor, D. Glick, Interscience Publishers, New York, 1957 p. 217-251.
- CLSI Guidelines and Standards, Clinical Laboratory Tests, AACC Press, Washington, 1990.
- Preeti Dalawari: emedicine.medscape.com/article/2090019-overview#a3.
- Burtis C.A., Ashwood E.R., ed. Tietz Textbook of Clinical Chemistry, 3rd ed. Philadelphia, PA: WB Saunders, 923, (1999).
- Polish National Chamber of Laboratory Diagnosticians, Oznaczenie etanolu w krwi i moczu pacjentów w laboratorium medycznym, Warszawa 2016. p.18-19.

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## ACCENT-200 ETHANOL

### • ACCENT MC240

Chem	ETH	No.	067	Sample Type	SERUM																																																								
Chemistry	ETHANOL	Print name	ETH	Reaction Direction	positive																																																								
Reaction Type	Kinetic	Sec Wave	505 nm	Decimals	0,1																																																								
Pri Wave	380 nm	Incubation Time		Reaction Time	3 7																																																								
Unit	mg/dl	Sample Vol		Aspirated		Diluent		Reagent Vol																																																					
Blank Time		Standard	2	µL	Decreased	2	µL	Increased		µL	R1	200	µL	R2		µL																																													
		Decreased		µL	Increased		µL	Sample Blank		Auto Rerun																																																			
Linearity range (Standard)	7	900	Linearity Limit	0.2	Linearity Range (Decreased)		Substrate Depletion	35000	Linearity Range (Increased)		Mixed Blank Abs	-35000	35000	R1 Blank Abs	-35000	35000	On-board Stability		Day(s)	Reagent Alarm Limit		Twin Chemistry		Enzyme Linear Extension		Prozone Check		Q1		Q2		V1		Q3		Q4		V2		Q5		Q6		V3		PC1		PC2		Sample Pretreatment		Control Pretreatment		Calibrator Pretreatment		Pretreat Sample Vol		µL	Pretreat Sample Vol		µL
<b>CALIBRATION SETTINGS</b>		<b>AUTO CALIBRATION</b>																																																											
Math model	Multi-point Linear		Bottle Changed																																																										
Factor		Replicates	2		Lot Changed																																																								
<b>ACCEPTANCE LIMITS</b>		<b>AUTO CALIBRATION</b>																																																											
Cal Time		Hour			Cal Time																																																								
Slope Diff		SD			Cal Time																																																								
Sensitivity		Repeatability	35000		Cal Time																																																								
Deter Coeff					Cal Time																																																								

### • ACCENT M320

Chem	ETH	No.	067	Sample Type	SERUM																																																								
Chemistry	ETHANOL	Print name	ETH	Reaction Direction	positive																																																								
Reaction Type	Kinetic	Sec Wave	505 nm	Decimals	0,1																																																								
Pri Wave	340 nm	Incubation Time	0	Reaction Time	3 7																																																								
Unit	mg/dl	Sample Vol		Aspirated		Diluent		Reagent Vol																																																					
Blank Time		Standard	2	µL	Decreased	2	µL	Increased		µL	R1	200	µL	R2		µL																																													
		Decreased		µL	Increased		µL	Sample Blank		Auto Rerun																																																			
Linearity range (Standard)	7	900	Linearity Limit	0.2	Linearity Range (Decreased)		Substrate Depletion	35000	Linearity Range (Increased)		Mixed Blank Abs	-35000	35000	R1 Blank Abs	-35000	35000	On-board Stability		Day(s)	Reagent Alarm Limit		Twin Chemistry		Enzyme Linear Extension		Prozone Check		Q1		Q2		V1		Q3		Q4		V2		Q5		Q6		V3		PC1		PC2		Sample Pretreatment		Control Pretreatment		Calibrator Pretreatment		Pretreat Sample Vol		µL	Pretreat Sample Vol		µL
<b>CALIBRATION SETTINGS</b>		<b>AUTO CALIBRATION</b>																																																											
Math model	Multi-point Linear		Bottle Changed																																																										
Factor		Replicates	2		Lot Changed																																																								
<b>ACCEPTANCE LIMITS</b>		<b>AUTO CALIBRATION</b>																																																											
Cal Time		Hour			Cal Time																																																								
Slope Diff		SD			Cal Time																																																								
Sensitivity		Repeatability	35000		Cal Time																																																								
Deter Coeff					Cal Time																																																								

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