Protein Calibrator Low

Liquid human based calibrator for use as a calibrator in immunoturbidimetric assays

REF	Cont.	
A00701	1x 1 mL	Protein Calibrator Low
A00702	1x 5 mL	Protein Calibrator Low

COMPOSITION

The Protein Calibrator Low is a defibrinated human plasma, liquid stabilized and filtered through a 0.2 µ pore-size filter.
 Sodium azide 0.095 %

PREPARATION

The Calibrator is liquid, ready to use.

STABILITY AND STORAGE

Storage: at 2 – 8 °C
 Stability: up to the expiration date
 Stability after opening: 6 weeks at 2 – 8 °C
 DO NOT FREEZE!

DIRECTIONS FOR USE

- Use this Calibrator in the same manner as specified with the reagent and/or instrument being used.
- Allow the Calibrator to equilibrate to room temperature (22 – 28 °C) prior to use.
- The Calibrator should not be allowed to stand for repeated long periods of time (up to 6 hours) at room temperature. Keep vials tightly capped at all times when not in use to avoid microbial contamination. Keep vials refrigerated when not in use

WARNINGS AND PRECAUTIONS

- For in Vitro Diagnostic use.
- Each individual donation intended for use in manufacture of protein calibrator was tested for hepatitis B surface antigen (HBsAg), anti-hepatitis C (anti-HCV) and anti-HIV 1 and HIV 2 by FDA required tests. Since no test method can assure that products derived from human blood do not contain HIV-1/2 and Hepatitis B and Hepatitis C virus, this material and all patient samples should be handled as though capable of transmitting infectious diseases.
- Reagents containing sodium azide must be handled with due caution: Do not ingest or allow to contact skin or mucous membranes! Sodium azide can form explosive azides when contacting heavy metals such as copper or lead.

WASTE MANAGEMENT


Please refer to local legal requirements.

LOT SPECIFIC ASSAY DATA

Values and expiry date are lot specific.

LOT: MPL007	EXP: 2017/02
Parameter	Assigned Value *
Albumin	5,963.0 mg/dL
α-1 Antitrypsin	201.0 mg/dL
α-2 Macroglobulin	380.0 mg/dL
α-1 Acid Glycoprotein	113.0 mg/dL
Ceruloplasmin	51.0 mg/dL
Haptoglobin	176.0 mg/dL
Transferrin	401.0 mg/dL
IgA	286.0 mg/dL
IgG	1,448.0 mg/dL
IgM	180.0 mg/dL
Complement C3	218.0 mg/dL
Complement C4	37.00 mg/dL
Antithrombin III	41.00 mg/dL
C1 Esterase Inhibitor **	41.00 mg/dL
Kappa Light Chain **	357.0 mg/dL
Lambda Light Chain **	194.0 mg/dL
Prealbumin	38.0 mg/dL

* Values traceable to ERM-DA470k from IFCC.
 ** Values based on Siemens standard material.

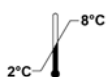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