## ASO Calibrator High

Liquid human based calibrator for use as a calibrator in immunoturbidimetric assays for ASO

REF	Cont.	
A00727	1x 1 mL	ASO Calibrator High
A00728	1x 5 mL	ASO Calibrator High

### COMPOSITION

Dialab ASO Calibrator High is based on human serum, which contains high levels of ASO with phosphate buffered saline containing 1 g % bovine serum albumin.

Sodium azide 0.095 %

### PREPARATION

The ASO Calibrator High is liquid, ready to use.

### STABILITY AND STORAGE

Storage: at 2 – 8 °C  
 Stability: up to the expiration date  
 Stability after opening: 6 weeks at 2 – 8 °C  
 DO NOT FREEZE!

### DIRECTIONS FOR USE

- Use this Calibrator in the same manner as specified with the reagent and/or instrument being used.
- Allow the Calibrator to equilibrate to room temperature (22 – 28 °C) prior to use.
- The Calibrator should not be allowed to stand for repeated long periods of time (up to 6 hours) at room temperature. Keep vials tightly capped at all times when not in use to avoid microbial contamination. Keep vials refrigerated when not in use.

### WARNINGS AND PRECAUTIONS

- For in Vitro Diagnostic use.
- Each individual donation intended for use in manufacture of protein calibrator was tested for hepatitis B surface antigen (HBsAg), anti-hepatitis C (anti-HCV) and anti-HIV 1 and HIV 2 by FDA required tests. Since no test method can assure that products derived from human blood do not contain HIV-1/2 and Hepatitis B and Hepatitis C virus, this material and all patient samples should be handled as though capable of transmitting infectious diseases.
- Reagents containing sodium azide must be handled with due caution: Do not ingest or allow to contact skin or mucous membranes! Sodium azide can form explosive azides when contacting heavy metals such as copper or lead.

### WASTE MANAGEMENT

Please refer to local legal requirements.

### LOT SPECIFIC ASSAY DATA

Values and expiry date are lot specific.

LOT: S2081	EXP: 2015/11
Parameter	Assigned Value
ASO *	193.0 IU/mL *
ASO **	318.0 IU/mL **
* traceable to WHO standardisation ** traceable to Dade Behring standardisation	

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