OnlyOne

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Data wydania: 05. 2022.

OnlyOne

Reagent name	Variant	Cat. No	Name	Cat. No
OnlyOne	500 ml	HM22-002-1	CLEANER FOR MYTHIC 22 DILUENT FOR MYTHIC 22 MYTHIC 22 MYTHIC 22AL MYTHIC 60 MYTHIC 70	HM22-001-1 HM22-003-10 N/A N/A N/A N/A

INTENDED USE
OnlyOne is intended to lyse of erythrocytes for hemoglobin release to allow further hemoglobin's assay on hematological analyzers: MYTHIC 22, MYTHIC 22AL, MYTHIC 60 and MYTHIC 70. It prepares leukocytes for subpopulations differentiation process. It is only for *in vitro* diagnostics, for healthcare professional users.

OPERATION PRINCIPLE

OnlyOne is a lysing reagent located on board of the analyzer. Due to the lysing properties of the reagent, it is possible to disrupt the cell membrane of red blood cells and release cellular contents in preparation for further analysis. OnlyOne contains surfactants that lyses the red blood cells and leukoprotective agents that protects the state of the leukocytes to permit the differentiation in subpopulations.

CONCENTRATION OF THE ACTIVE INGREDIENTS IN THE REAGENT

saponin	0.1 – 2.0 g/l
detergents	0.1 – 2.0 g/l
quaternary ammonium salt	0.05 – 0.5 g/l
glutaraldehyde	< 0.1 %
mixture of 5-chloro-2-methyl-4-isothiazolin-3-one and 2-methyl-4-isothiazol-3-one (3:1)	< 0.0015 %
sodium chloride	
buffer	
anticoagulant	

REAGENT STABILITY

The reagent must be used before the expiry date indicated on the label.
Once opened, the reagent is stable for 60 days at room temperature.

WARNING AND NOTES

- Do not use reagent beyond expiry date printed on the label.
- Do not combine the remaining reagent with the reagent from a new package.
- Use personal protective equipment to prevent contact with samples, reagents and controls.
- Avoid discharge to sewage system or to environment.
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.
- EUH 208 OnlyOne contains reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one and 2-methyl-2H-isothiazol-3-one (3:1). May produce an allergic reaction.
- EUH210 Safety data sheet available on request.

DEVICES INTENDED TO BE USED IN COMBINATION WITH THE REAGENT

OnlyOne is intended to be used in combination with the cleaner, diluent and appropriate hematology analyzer.

ASSAY PROCEDURE

OnlyOne is ready for use.
Follow the analyzer's user manual.
Prior the first use, unopened reagent must be stored for minimum 24 hours at room temperature.

WASTE MANAGEMENT¹

After use, the reagents should be handled as potentially infectious and disposed of in accordance with local legal requirements.

- Reagents residues: 18 01 07
- Empty packages: 15 01 02
- Wastewater from the analyzer: 18 01 03*

INCIDENTS²

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (websites address: incidents@cormay.pl) and the competent authority of the Member State in which the user and/or the patient is established.

Serious incident means any incident that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- a serious public health threat.

LITERATURE

- European Commission notice on technical guidance on the classification of waste (2018/C 124/01) of 9 April 2018.
- Regulation (EU) 2017/746 of the European Parliament and of the Council of 5 April 2017 on in vitro diagnostic medical devices.

LIST OF CHANGES

Previous version: 01	Current version: 02
Sections updated: <i>INTENDED USE; CONCENTRATION OF THE ACTIVE INGREDIENTS IN THE REAGENT; WARNING AND NOTES; DEVICES INTENDED TO BE USED IN COMBINATION WITH THE REAGENT; SYMBOL EXPLANATION</i>	
Information added: <i>AUTHORIZED REPRESENTATIVE</i>	

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PZ CORMAY S.A.