## DILUENT FOR MYTHIC 22

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### INTENDED USE

DILUENT FOR MYTHIC 22 is intended to dilute blood samples before assay and to maintain appropriate environment during assay on hematology analyzers MYTHIC 22, MYTHIC 22AL, MYTHIC 60 and MYTHIC 70. It is only for *in vitro* diagnostics, for healthcare professional users.

### OPERATION PRINCIPLE

DILUENT FOR MYTHIC 22 is isotonic reagent connected to the analyzer. By diluting the blood sample, it allows it to flow through the measuring aperture of the analyzer and to make a measurement.

### CONCENTRATION OF THE ACTIVE INGREDIENTS IN THE REAGENT

2-phenoxyethanol	0.022 %
buffer	
sodium chloride	
potassium chloride	
EDTA	
stabilizer	
detergent	

### REAGENT STABILITY

The reagent must be used before the expiry date indicated on the label. Once opened, the reagent is stable for 60 days at room temperature.

### WARNING AND NOTES

- Do not use the reagent beyond expiry date printed on the label.
- Do not combine the remaining reagent with the reagent from a new package.
- Use personal protective equipment to prevent contact with samples, reagents and controls.
- Avoid discharge to sewage system or to environment.
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.

### DEVICES INTENDED TO BE USED IN COMBINATION WITH THE REAGENT

DILUENT FOR MYTHIC 22 reagent is intended to be used in combination with the lysing reagent, cleaner and appropriate hematology analyzer.

Name	Cat. No
CLEANER FOR MYTHIC 22	HM22-001-1
OnlyOne	HM22-002-1
MYTHIC 22	N/A
MYTHIC 22AL	N/A
MYTHIC 60	N/A
MYTHIC 70	N/A

### ASSAY PROCEDURE

DILUENT FOR MYTHIC 22 is ready for use. Follow the analyzer's user manual.

Prior the first use, unopened reagent must be stored for minimum 24 hours at room temperature. The reagent container must be placed on the same level as the analyzer. Do not leave the reagent on the floor.

Prior the first use of DILUENT FOR MYTHIC 22:

- Mix reagent accurately but gently for about 1 minute rotating a container from vertical to horizontal position, to assure homogeneity of solution.
- Leave reagent to stand for 12 (at least) to 24 hours (optimally) to avoid air bubbles occurrence.
- Connect reagent to analyser.
- Follow the analyzer's operator manual.

### WASTE MANAGEMENT<sup>1</sup>

After use, the reagents should be handled as potentially infectious and disposed of in accordance with local legal requirements

- Reagents residues: 18 01 07
- Empty packages: 15 01 02
- Wastewater from the analyzer: 18 01 03\*

### INCIDENTS<sup>2</sup>

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (websites address: incidents@cornay.pl) and the competent authority of the Member State in which the user and/or the patient is established.

**Serious incident** means any incident that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- a serious public health threat.

### LITERATURE

- European Commission notice on technical guidance on the classification of waste (2018/C 124/01) of 9 April 2018.
- Regulation (EU) 2017/746 of the European Parliament and of the Council of 5 April 2017 on *in vitro* diagnostic medical devices.

### LIST OF CHANGES

Previous version: 01	Current version: 02
Sections updated: INTENDED USE; DEVICES INTENDED TO BE USED IN COMBINATION WITH THE REAGENT, SYMBOL EXPLANATION	
Information added: AUTHORIZED REPRESENTATIVE	

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**PZ CORMAY S.A.**