Liquid Standard

AMMONIA STANDARD

Standard for the use in tests for the quantitative in vitro determination of ammonia in human plasma on photometric systems

REF	Kit Size	Configuration
Y08310SV	1 x 5 mL	Single vial

COMPOSITION

Ammonium chloride in aqueous buffer, preservative.

STANDARD PREPARATION

The standard is liquid, ready to use.
 Allow the material to reach room temperature. Gently invert the vial and swirl to assure contents are homogenous. Avoid foaming.

STABILITY AND STORAGE

Storage: at 2 – 8 °C
 Stability (unopened): until the expiry date.
 Stability after opening: 2 weeks when stored tightly capped.

CLOSE IMMEDIATELY AFTER USE!

PREVENT EVAPORATION!

PROTECT FROM LIGHT AND HEAT.

Do not use the controls if there is visible evidence of microbial growth in the vial.

MATERIALS REQUIRED BUT NOT PROVIDED

Ammonia reagent
 Analyser and general laboratory equipment.

TEST PROCEDURE

Please refer to the reagent package inserts and to the user's manual of the analyser for instructions for use.
 If trends or sudden shifts in values are detected, review all operating parameters. Control recovery should lie within acceptable intervals. Results outside the specified values even after recalibration could be due to reagent/standard/control deterioration, unsuitable storage conditions, instrument malfunction, or error during test procedure. Each laboratory should establish guidelines for corrective actions to be taken if controls do not recover within the specified limits.

ASSIGNED VALUES

The assigned value IS lot specific. Please refer to the table below for the lot specific value.

WARNINGS AND PRECAUTIONS

1. This product is for in vitro diagnostic use only.
2. Contains sodium azide (NaN₃) < 0.1 %. Avoid contact with eyes, skin and mucous membranes. Flush waste pipes with water after the disposal of undiluted controls in order to avoid azide build up in the drain.
3. Unsuitable storage, handling or errors during the analytical procedure may give erroneous results.
4. Please refer to the safety data sheets and take the necessary precautions for the use of calibrators and controls.
5. For professional use only!

WASTE MANAGEMENT

Please refer to local legal requirements.

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